ACCU-CHEK®
Nano

Roche

Owner's Booklet for Self-Testing Only

Blood Glucose Meter

ACCU-CHEK®

Contents

Important Safety Information	1
Chapter 1: Your New System.....	5
Chapter 2: Control Tests.....	10
Chapter 3: The Accu-Chek FastClix Lancing Device	13
Chapter 4: Blood Glucose Tests.....	17
Chapter 5: Meter Memory, Setup, and Data Transfer.....	24
Chapter 6: Meter and Lancing Device Cleaning and Disinfecting.....	33
Chapter 7: Maintenance and Troubleshooting.....	38
Chapter 8: Technical Information.....	42
Index	47

ACCU-CHEK, ACCU-CHEK NANO, ACCU-CHEK SMARTVIEW,
ACCU-CHEK NANO SMARTVIEW, and FASTCLIX are trademarks of Roche.
All other product names and trademarks are the property of their respective owners.

© 2021 Roche Diabetes Care

06333630006-0721

Roche Diabetes Care, Inc.
9115 Hague Road
Indianapolis, IN 46256
www.accu-chek.com

The Accu-Chek Nano SmartView System

The Accu-Chek Nano SmartView blood glucose monitoring system is intended to be used for the quantitative measurement of glucose (sugar) in fresh capillary whole blood samples drawn from the fingertips or palm. The Accu-Chek Nano SmartView blood glucose monitoring system is intended to be used by a single person and should not be shared.

The Accu-Chek Nano SmartView blood glucose monitoring system is intended for self-testing outside the body (in vitro diagnostic use) by people with diabetes at home as an aid to monitor the effectiveness of diabetes control. The Accu-Chek Nano SmartView blood glucose monitoring system should not be used for the diagnosis of or screening of diabetes or for neonatal use. Alternate site testing should be done only during steady-state times (when glucose is not changing rapidly).

The Accu-Chek SmartView test strips are for use with the Accu-Chek Nano blood glucose meter to quantitatively measure glucose (sugar) in fresh capillary whole blood samples drawn from the fingertips or palm.

This system is intended to be used by a single person and should not be shared.

The Accu-Chek Nano SmartView blood glucose monitoring system includes:

Accu-Chek Nano meter with batteries, Accu-Chek SmartView test strips*, Accu-Chek SmartView control solution*, Accu-Chek FastClix lancing device* (with a black cap for fingertip testing and an AST cap** for alternate site testing), Accu-Chek FastClix lancet drum*.

*may be sold separately

**available upon request from the Accu-Chek Customer Care Service Center at 1-800-858-8072

WARNING

Choking hazard. Small parts. Keep away from children under the age of 3 years.

NOTE

The term “blood glucose” is used when referring to “blood sugar.”

Important Safety Information

Need Help?

Contact the Accu-Chek Customer Care Service Center toll-free at 1-800-858-8072. We offer assistance 24 hours a day, 365 days a year in many languages. You can also visit accu-chek.com for diabetes management tools and product demonstrations.

Please complete the warranty card and mail it, so you receive the best customer service possible and product update news.

About Testing Yourself or Others

WARNING

- DO NOT CHANGE YOUR THERAPY BASED ON A TEST RESULT THAT DOES NOT MATCH HOW YOU FEEL OR IF YOU BELIEVE THAT YOUR TEST RESULT COULD BE INCORRECT.
- It is always a good idea to have a back-up testing method available. Failure to test could cause a delay in therapy decisions and lead to a serious medical condition. Examples of back-up testing methods include a back-up meter or testing by a laboratory. Ask your healthcare professional or pharmacist about other possible back-up methods.
- If your blood glucose result does not match how you feel and you have followed the instructions in this Owner's Booklet, follow your healthcare professional's instructions, or contact your healthcare professional.

Important Safety Information

WARNING

- During normal testing, any blood glucose meter or lancing device may come in contact with blood. All parts of the kit are considered biohazardous and can potentially transmit infectious diseases from bloodborne pathogens, even after you have performed cleaning and disinfecting.^{1,2}
- The meter and lancing device should never be used by more than one person. Do not share the meter and lancing device with anyone, including family members, due to the risk of infection from bloodborne pathogens.^{1,2}
- Cleaning and disinfecting the meter and lancing device destroys most, but not necessarily all, bloodborne pathogens.³
- If the meter is being operated by a second person who is providing testing assistance to the user, the meter and lancing device should be cleaned and disinfected prior to use by the second person.
- Disinfect the meter and lancing device before allowing anyone else to handle them. Do not allow anyone else to test with the meter or lancing device.
- It is important to keep the meter and lancing device clean and disinfected. For instructions on how to clean and disinfect the meter and lancing device, see Chapter 6, Meter and Lancing Device Cleaning and Disinfecting.
- Wash hands thoroughly before and after handling the meter, lancing device, or test strips.
- Failure to follow testing instructions or test strip storage and handling instructions can lead to an incorrect test result that may lead to improper therapy. Carefully read and follow the instructions in the User's Manual and package inserts for the test strips and control solutions.
- Inspect the test strip container before using the test strips for the first time. If you see any damage to the container, if anything prevents the cap from closing properly, or if the container was open before using for the first time, do not use the test strips. Do not perform a control test. Contact the Accu-Chek Customer Care Service Center. Damaged test strips can cause inaccurate results, which could lead to improper therapy.
- The meter, test strips, and control solution are only for use outside the body (in vitro). Do not eat the test strips. Do not swallow or inject the control solution or use the control solution for any purpose other than testing the Accu-Chek Nano SmartView system.

NOTE

- After inspecting the container per instructions above, perform a control test whenever opening a new test strip box. Performing a control test lets you know that the meter and test strips are working properly.
- Refer to the test strip and control solution package inserts for additional health-related information.

Important Safety Information

Special Information for Caregivers

- Do not use the meter system to measure blood glucose in people who are experiencing cardiovascular collapse (severe shock) or decreased peripheral blood flow.
- Consult your healthcare professional to determine if it is appropriate for your child to be taught how to use the meter system or any other medical products.
- Some people with diabetes do not experience symptoms of low blood glucose (hypoglycemia). Others, such as children or people who are unconscious or have certain disabilities, may not be able to communicate their symptoms to caregivers. For these reasons, do not change any therapy without first talking to a healthcare professional.

Before You Start Testing

NOTE

Set the time and date on the meter before you begin testing.

The Accu-Chek Nano Meter

1. Display

Shows results, messages, and test results stored in memory.

2. Test Strip Slot

Insert test strip here.

3. Battery Drawer

Pull out to replace batteries.

4. Power/Set Button

Turns meter on or off and sets options.

5. Right Arrow and Left Arrow Buttons

Press to enter memory, adjust settings, and scroll through test results.

6. Infrared (IR) Window

Transfers data from the meter to a computer.

1

Your New System

(For Example)

11

7. Test Strip Container

8. Insert this end into meter.

9. Yellow Window

Touch blood drop or control solution here.

10. Control Solution Bottle

11. Batteries

The Accu-Chek FastClix Lancing Device

1. Cap

For fingertip testing.

2. Comfort Dial with Depth Selection

Select penetration depth.

3. Penetration Depth Indicator

Points to depth setting.

4. Lever

Advance to a new lancet.

5. Lancet Counter Window

Shows number of available lancets.

6. Release button

Press to prime and prick.

7. Lancet Drum

Contains 6 lancets.

8. AST Cap

For Alternate Site Testing (available upon request from the Accu-Chek Customer Care Service Center at 1-800-858-8072).

7

8

NOTE

Some items may be sold separately.

Using the Accu-Chek Nano SmartView System

- Use only Accu-Chek SmartView test strips.
- Use the test strip immediately after removing it from the test strip container.
- Do not apply blood or control solution to the test strip before inserting it into the meter. Retest with a new test strip. If a result appears before applying blood or control solution, do not act on that result.
- Close the test strip container tightly immediately after removing a test strip. Moisture can damage the test strips and produce incorrect results.
- Store the unused test strips in their original container with the cap tightly closed.
- Discard the test strips if they are past the **Use By** date printed on the test strip container. If the **Use By** date is missing or cannot be read, do not use the test strips. Contact the Accu-Chek Customer Care Service Center at 1-800-858-8072.
- Refer to the test strip package insert for test strip storage and system operating conditions.
- Do not remove test strips from the test strip container and put them into another container, such as a plastic bag, pocket, purse, wallet, etc.
- Do not reuse test strips. Once control solution or blood has been applied to a test strip, discard it. If a retest is necessary, use a new test strip.
- Perform a control test every time you open a new test strip box.

WARNING

To prevent inaccurate results:

- DO NOT expose test strips to heat, moisture, or humidity. Temperatures outside the required range, as well as moisture and humidity, can damage the test strips and lead to inaccurate results.
- DO NOT bend, cut, or alter the test strips.
- DO NOT get dirt, food, or other material on the test strip.

Button Functions

Here are the functions of the power/set and arrow buttons on the meter. These functions are used throughout this booklet.

Button	Function	Action
	Turn the meter on or off	Press and release  .
	Enter the set-up mode	Turn the meter on. Press and hold  until set-up appears on the display.
	Exit the set-up mode at any time	Press and hold  until the flashing test strip symbol appears on the display.
	Set the chosen feature	Press and release  .
	Check the display segments	<p>Check the display before each test to make sure all segments are clear and look like the picture to the left.</p>  <p>Turn the meter off. Press and hold  to see the complete display.</p> <p>If one of the segments is missing or looks different from the picture, do not use the meter. Contact the Accu-Chek Customer Care Service Center at 1-800-858-8072.</p>
 	Adjust settings for time and date, beeper, test reminders, and hypoglycemic alert	<p>Press and release  or .</p> <p>Press and hold  or  to scroll faster.</p>
	Enter memory	Press and release  or  .
	Mark a blood glucose result	Press and release  .

When to Perform a Control Test

Performing a control test lets you know the meter and test strips are working properly. You should perform a control test when:

- you open a new test strip box.
- you think the test strips are damaged.
- you want to check the meter and test strips.
- the test strips were stored in extreme temperatures, humidity, or both.
- you dropped the meter.
- your test result does not match how you feel.
- you want to check if you are performing the test correctly.

About the Control Solution

- Use only Accu-Chek SmartView control solution.
- Close the control solution bottle tightly after use.
- Write the date you open the control solution bottle on the bottle label. The control solution must be discarded 3 months from the date the control solution bottle was opened (discard date) or on the **Use By** date on the bottle label, whichever comes first.
- Do not use control solution that is past the **Use By** or discard date.
- Refer to the control solution package insert for control solution storage conditions.
- The meter automatically recognizes the difference between the control solution and blood.
- The control results are not displayed in memory.
- The control solution can stain fabric. Remove stains by washing with soap and water.
- Control solution is available for purchase. To order the control solution, talk to your pharmacist or visit accu-chek.com to order online.

Performing a Control Test

You need the meter, a test strip, and control solution.

1

Check the meter display for missing segments. All segments should look like the picture.

2

Check the **Use By** date on the test strip container. Do not use test strips past the **Use By** date.

3

Insert the test strip into the meter in the direction of the arrows. Place the meter on a flat surface.

4

Remove the bottle cap. Wipe the tip of the bottle with a tissue. Squeeze the bottle until a tiny drop forms at the tip.

5

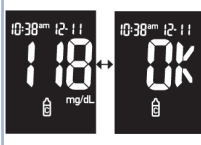
Touch the drop to the **front edge** of the yellow window of the test strip until you see  flash. **Do not put control solution on top of the test strip.**

When you see  flash, there is enough control solution in the test strip.

The control result and the bottle symbol appear on the display.

Wipe the tip of the bottle with a tissue. Cap the bottle tightly. **OK** and the control result alternate on the display if the control result is in range **OR Err** and the control result alternate on the display if the control result is not in range.

6

Remove and discard the used test strip. After a successful test, the meter turns off 5 seconds after the test strip is removed.

Understanding Out-of-Range Control Results

WARNING

The control range applies only to a control result. It only indicates that the test strips and meter are working properly. Do not use a control result to interpret blood glucose results.

If the control result is out of range, do not use the meter until you solve the problem. Check this list to help solve the problem.

Troubleshooting Checks	Action
1. Were the test strips or control solutions expired?	Discard the test strips or control solution if either is past the Use By date. If the control solution was opened more than 3 months ago, discard it. Repeat the control test with an unexpired test strip and an unexpired control solution.
2. Did you wipe the tip of the control solution bottle before use?	Wipe the tip of the bottle with a tissue. Repeat the control test with a new test strip and a fresh drop of control solution.
3. Were the caps on the test strip container and the control solution bottle always closed tightly?	Replace the test strips or control solution if you think either was uncapped for some time. Repeat the control test.
4. Was the test strip used immediately after it was removed from the test strip container?	Repeat the control test with a new test strip and a fresh drop of control solution.
5. Were the test strips and control solutions stored in a cool, dry place?	Repeat the control test with a properly stored test strip or control solution.
6. Did you follow the directions?	Read Chapter 2, Control Tests, and repeat the control test.
7. Are you still unsure of the problem?	Contact the Accu-Chek Customer Care Service Center at 1-800-858-8072.

Using the Accu-Chek FastClix Lancing Device

- The black cap is for fingertip testing only.
- Remember that the opening where the lancet comes out is not in the center of the cap.
- Remove the cap by pulling it straight off. Do not twist the cap.

- The AST cap* is for testing other approved sites on the body, not the fingertip.
- Remove the cap by pulling it straight off. Do not twist the cap.
- For palm testing, we suggest a depth setting of 3. Once you perform a successful blood glucose test, you may want to find the lowest depth setting that provides enough blood with the least amount of pain.

- A new lancet drum is dark gray with a white end.

- A used lancet drum has a visible red stripe on the white end.
- You cannot reuse a used lancet drum.
- Do not remove the lancet drum until you have used all 6 lancets.

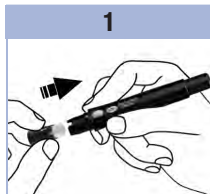
- The penetration depth indicator on the comfort dial shows the current depth setting. The higher the number, the deeper the penetration. The best depth setting is the lowest number that lets you get enough blood for a test. Try different depth settings to find the one that is right for you.
- For soft skin, we suggest a depth setting of 2. For thick skin, try a higher depth setting.

*Available upon request from the Accu-Chek Customer Care Service Center at 1-800-858-8072.


3 The Accu-Chek FastClix Lancing Device

Inserting a Lancet Drum

You must first load the lancet drum into the lancing device to get it ready for use.

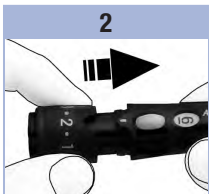Remove the cap. Insert a new lancet drum, white end first, until it clicks firmly into place.

Slide the cap on until it stops by aligning the notch on the cap with the notch on the lancing device.

You are now ready to use the first lancet. The lancet counter shows a number 6, meaning you have six new lancets remaining.

NOTE

Once the lancet drum is inserted into the lancing device, do not remove the lancet drum until completely used. The lancet drum cannot be reused once it has been removed from the lancing device.

WARNING

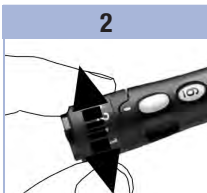
You must not insert the lancet drum into the lancing device and simultaneously press the release button or hold the lancing device with the release button resting on a surface such as a table top. This could release a lancet and inadvertently cause injury.

Using the Accu-Chek FastClix Lancing Device

1

Make sure you select the correct cap.

2


Adjust the lancet depth by turning the comfort dial.

3

Obtain a blood drop. See Chapter 4, Blood Glucose Tests.

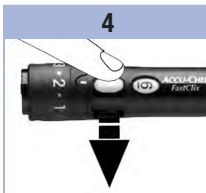4

Advance to the next lancet by sliding the lever forward and back all the way.

The number in the lancet counter decreases by 1. The lancet counter window shows the number of remaining lancets.

NOTE

- ALWAYS use a new, sterile lancet each time you test to avoid infection.
- For safety reasons, once you advance to a new lancet, you cannot go back to a used lancet.
- NEVER reuse a lancet.
- NEVER share your lancing device with anyone.

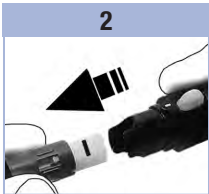
3 The Accu-Chek FastClix Lancing Device

Changing the Lancet Drum

When you have used the sixth and last lancet, change the lancet drum.

1


Remove the cap.

Hold the lancet drum between your thumb and index finger and pull it straight out.

A red stripe is visible on the white part, indicating that the lancet drum has been used.

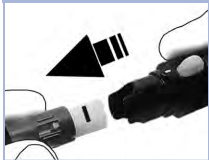2

Throw the old lancet drum away.

Always discard according to local regulations.

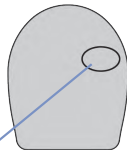
4

Insert a new lancet drum, white end first, until it clicks firmly into place. Slide the cap back on until it stops by aligning the notch on the cap with the notch on the lancing device.

WARNING

Blood glucose results can be displayed in either mg/dL or mmol/L. The back label of the meter shows the unit of measurement. If the meter shows the wrong unit, contact the Accu-Chek Customer Care Service Center at 1-800-858-8072. If you do not know which unit of measurement is correct for you, contact your healthcare professional. Using the wrong unit of measurement may cause misinterpretation of your actual blood glucose level and may lead to improper therapy.

mg/dL or mmol/L
is printed here

Performing a Blood Glucose Test with Blood from Your Fingertip

Refer to the Important Safety Information at the beginning of this booklet.

Before you perform your first blood glucose test, set up the meter correctly and perform a control test. You need the meter, a test strip, and a lancing device with a lancet drum loaded to perform a blood test.

1

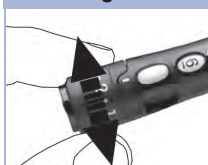
Check the meter display for missing segments. All segments should look like the picture.

2

Wash your hands with warm soapy water and dry thoroughly. Dirty or wet hands could affect test results.

3

Adjust the comfort dial.

4

Check the **Use By** date on the test strip container. Do not use test strips past the **Use By** date.

5

Insert the test strip into the meter in the direction of the arrows. The meter turns on and beeps.

6

When the blood drop symbol flashes, obtain a blood drop.

7

Place the lancing device firmly against the side of your fingertip. Press the release button all the way down to prick your finger.

8

Gently squeeze your finger to assist the blood flow. This helps you get a blood drop.

9

Touch the blood drop to the **front edge** of the yellow window of the test strip.

Do not put blood on top of the test strip.

The meter beeps and  flashes when there is enough blood in the test strip.

10

The test result appears on the display.

To assign a marker to the test result, leave the test strip in the meter.

See Chapter 4, Marking Blood Glucose Results and Setting the Post-Meal Test Reminder.

Otherwise, remove and discard the used test strip. After a successful test, the meter turns itself off 5 seconds after the test strip is removed.

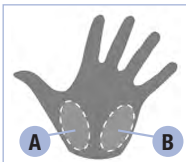
11

Advance to the next lancet.

Wash hands thoroughly with soap and water.

Performing a Blood Glucose Test with Blood from Your Palm (Alternate Site Testing)

You have the option of obtaining a blood sample from other sites on your body besides the fingertip. Alternate sites include 2 palm testing sites. The 2 palm testing sites are the fleshy areas under the thumb (A thenar) and under the little finger (B hypothenar).

Blood obtained from the fingertip can be used at any time to test blood glucose. If blood from an alternate site is used, there are certain times when testing is not appropriate. This is because your blood glucose level changes faster in your fingertip than in the alternate sites. These differences may cause you to misinterpret your actual blood glucose level, leading to improper therapy and potential adverse health effects. Read the next section before you try testing from alternate sites.

<p>You may perform a palm test</p>	<ul style="list-style-type: none"> • immediately before a meal. • while fasting.
<p>You may NOT perform a palm test</p>	<ul style="list-style-type: none"> • up to 2 hours following a meal, when blood glucose values can rise quickly. • after injecting bolus insulin, when blood glucose values can decrease rapidly. • after exercise. • if you are sick. • if you think your blood glucose is low (hypoglycemia). • if you sometimes do not notice when your blood glucose is low.

If you are interested in AST, talk to your healthcare professional first.

To obtain an AST cap and detailed AST instructions, contact the Accu-Chek Customer Care Service Center at 1-800-858-8072.

Marking Blood Glucose Results and Setting the Post-Meal Test Reminder

You can mark a blood glucose result to indicate a special event. If you select a test result marker, it is automatically stored in memory. When you review the test results in memory, these markers can help you remember what was different about the test result.

Symbol	Function
	Pre-Meal Marker Marking blood glucose results with a pre-meal marker provides more information about your test results to help you and your healthcare professional in the management of your diabetes.
	Pre-Meal Marker with Post-Meal Test Reminder When a pre-meal result is marked with a post-meal test reminder, the meter beeps 1 or 2 hours after you test to remind you to do a post-meal test. Set the post-meal test reminder time to 1 or 2 hours in the set-up mode. The meter will beep 1 or 2 hours after the pre-meal test to remind you to do a post-meal test.
	Post-Meal Marker Marking blood glucose results with a post-meal marker provides more information about your test results to help you and your healthcare professional in the management of your diabetes. The post-meal marker automatically appears with the test result if a test is performed 15 minutes before or after the post-meal test reminder is programmed to beep.
	General Marker You might want to use the general marker to mark an event such as an AST result or exercise.

Here is how to mark a test result and initiate a post-meal test reminder:

1. Perform a blood glucose test.
The blood glucose results appears on the display.
2. Leave the test strip in the meter. Press and release  to toggle through the test result markers and post-meal test reminder.
3. When the marker or test reminder that you wish to select appears on the display, remove the test strip from the meter.

Unusual Blood Glucose Results

If your blood glucose result does not match how you feel, follow these steps:

1. Perform a control test. See Chapter 2, Control Tests.
2. Repeat the blood glucose test. See Chapter 4, Performing a Blood Glucose Test with Blood from Your Fingertip.

If your blood glucose result still does not match how you feel, follow your healthcare professional's instructions or call your healthcare professional immediately.

NOTE

Always follow your healthcare professional's instructions. For example, if your healthcare professional has advised you to immediately treat a low blood glucose result (such as by eating something), then do that first.

WARNING

- Do not change your treatment because of one blood glucose result.
- NEVER ignore symptoms of low or high blood glucose.

Symptoms of Low or High Blood Glucose

WARNING

The meter is designed to provide a numerical value for blood glucose in the range of 20–600 mg/dL. If you receive a numerical value below 20 mg/dL or above 600 mg/dL and it does not match how you feel, contact the Accu-Chek Customer Care Service Center toll-free at 1-800-858-8072.

Being aware of the symptoms of low or high blood glucose can help you understand your test results and decide what to do if they seem unusual.

Low blood glucose (hypoglycemia): Symptoms of hypoglycemia may include, but are not limited to, anxiety, shakiness, sweating, headache, increased hunger, dizziness, pale skin color, sudden change in mood or irritability, fatigue, difficulty concentrating, clumsiness, palpitations, and/or confusion.

High blood glucose (hyperglycemia): Symptoms of hyperglycemia may include, but are not limited to, increased thirst, frequent urination, blurred vision, drowsiness, and/or unexplained weight loss.

If you are experiencing any of these symptoms, or other unusual symptoms, test your blood glucose from the fingertip. If your blood glucose result is displayed as LO or HI, follow your healthcare professional's instructions or contact your healthcare professional immediately. If your blood glucose result does not match how you feel, follow the steps under Chapter 4, Unusual Blood Glucose Results.

Comparing Your Meter Result to a Laboratory Result

A common question is how the blood glucose results on the meter compare to the laboratory results. Your blood glucose can change quickly, especially after eating, taking medication, or physical activity. If you test yourself in the morning, then go to your healthcare professional's office for a blood glucose test, your test results will probably not match, even if you are fasting. This is typically not a problem with the meter, it just means that time has elapsed and your blood glucose level has changed.

Although you always apply fresh capillary whole blood to the test strip, the system has been calibrated to deliver plasma-like values for easier comparison to laboratory results.

If you want to compare your meter result to the laboratory result, **you must be fasting**. Take the meter to your healthcare professional's office and test yourself by fingerstick within 5 minutes of having blood drawn from your arm by a healthcare professional. Keep in mind that the laboratory uses different technology than the meter and that blood glucose meters for self-testing generally read somewhat lower than the laboratory result.

Storing Blood Glucose and Control Results

The meter automatically stores up to 500 blood glucose results with the time and date of the test result and any test result marker which you can review at any time. Test results are stored from the newest to the oldest, so set the time and date correctly in the meter. Having the correct time and date setting helps ensure appropriate interpretation of stored blood glucose results by you and your healthcare team.

NOTE

- The memory is not lost when the batteries are replaced; however, confirm that the time and date are still correct. See Chapter 5, Setting the Time and Date.
- Once 500 blood glucose or 20 control results are in memory, adding a new result causes the oldest one to be deleted.
- If more than 500 blood glucose tests are performed within a 90-day period, only the most recent 500 test results are included in the 90-day average.
- The control results are stored in memory, but cannot be reviewed on the meter. To view stored control results, first transfer them to a compatible software application. For product availability, contact the Accu-Chek Customer Care Service Center at 1-800-858-8072.
- Control results are not included in the 7-, 14-, 30-, and 90-day averages.
- Only test results that have been assigned a pre-meal or post-meal marker are included in pre-meal and post-meal averages. All blood glucose results are included in the general 7-, 14-, 30-, and 90-day averages.

Viewing Test Results in Memory

With the meter on or off, press and release ◀ or ▶ to enter memory. The most recent test result appears on the display.

Button	Function	Action
◀	Previous test results	<p>Press ◀ to view previous test results from newest to oldest.</p> 
▶	General averages	<p>Press ▶ to view 7-, 14-, 30-, and 90-day averages. n = number of test results in the average.</p> 
▶	Pre-meal averages Post-meal averages	<p>Continue to press ▶ to view 7-, 14-, 30-, and 90-day pre-meal and post-meal averages. Only test results with a pre-meal or post-meal marker are included in the pre-meal or post-meal averages.</p>

WARNING

Do not change your therapy based on an individual test result in memory. Talk to your healthcare professional before changing therapy based on test results in memory.

5 Meter Memory, Setup, and Data Transfer

Using the Set-Up Mode

The following features can be customized as required.

Symbol	Function	Action	Factory Pre-set
	Time and date	Set the time and date.	
	Beeper	Select On or OFF . Setting the beeper to OFF does not affect test results. The beeper must be On to use the test reminder feature.	<i>ON</i>
	Post-meal test reminder	Select 1 hour or 2 hours.	<i>2Hr</i>
	Test reminders	Select On or OFF . You can set up to 4 test reminders per day.	A-1 8:00 am A-2 12:00 pm (noon) A-3 6:00 pm A-4 10:00 pm
	Hypo alert	Select On or OFF . The hypo alert can be set to a level between 50 and 90 mg/dL to let you know when your blood glucose is possibly too low.	<i>OFF</i>

Setting the Time and Date

1

Turn the meter on.
The flashing test strip symbol appears on the display.

2

Press and **hold**  until **set-up** appears on the display.

The hour flashes.

3

Press and release  or  to decrease or increase the hour.

Press and release  to set the hour.

The minutes flash.

4

Repeat step 3 to set minutes, am/pm, month, day, and year.

To set up more options, press and release .

To exit, press and **hold**  until the flashing test strip symbol appears on the display.

Setting the Beeper On or OFF

NOTE

The beeper prompts you:

- to apply blood or control solution to the test strip.
- when enough blood or control solution is drawn into the test strip.
- when the blood glucose or control test is complete.
- when a button is pressed.
- when it is time to perform a test (if you set the test reminders or post-meal test reminder).
- if an error occurred while performing a blood glucose or control test (even if the beeper is off, it still beeps for an error).

1

Turn the meter on.
The flashing test strip symbol appears on the display.
Press and **hold**  until **set-up** appears on the display.

2

Press and release  repeatedly until the flashing beeper symbol and **On** or **OFF** appear.

3

Press and release  or  to switch between **On** and **OFF**.
To set up more options, press and release .
To exit, press and **hold**  until the flashing test strip symbol appears on the display.

Setting the Post-Meal Test Reminder

NOTE

The post-meal test reminder:

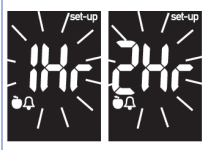
- beeps 1 or 2 hours after the blood glucose test to remind you to do a post-meal test.
- beeps every 2 minutes up to 3 times.
- turns off by inserting a test strip or pressing any button.

- If a test was performed within 15 minutes of a test reminder, the test reminder does not beep.
- If the meter is on at the test reminder time, the test reminder does not beep.
- Exposure to cold conditions may disable test reminders until the meter is turned on.

1

Turn the meter on.
The flashing test strip symbol appears on the display.
Press and **hold**  until **set-up** appears on the display.

2

Press and release  repeatedly until set-up, and the flashing **1Hr** or **2Hr** appear.

3

Press and release  or  to select **1Hr** or **2Hr**.
Press and release  to set the option.
To set up more options, press and release .
To exit, press and **hold**  until the flashing test strip symbol appears on the display.

4

This display appears when a post-meal test reminder occurs.

5 Meter Memory, Setup, and Data Transfer

Setting the Test Reminders

NOTE

Test reminders:

- beep at the same time each day.
- beep every 2 minutes up to 3 times.
- turn off by inserting a test strip or pressing any button.

- If a test was performed within 15 minutes of a test reminder, the test reminder does not beep.
- If the meter is on at the test reminder time, the test reminder does not beep.
- Exposure to cold conditions may disable test reminders until the meter is turned on.

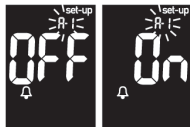
1

Turn the meter on.

The flashing test strip symbol appears on the display.

Press and **hold**  until **set-up** appears on the display.

2

Press and release  repeatedly until the bell symbol, **OFF**, **set-up**, and the flashing **A-1** appear.

3

Press and release  or  to switch between **On** and **OFF**.

Press and release  to set your choice.

If **On** is selected, the hour flashes.

Press and release  or  to adjust the hour.

Press and release  to set the hour.

4

The minutes flash.

Press and release  or  to select **00**, **15**, **30**, or **45**. These are the only choices.

Press and release  to set the minutes.

Am or **pm** flashes on the display.

5

Press and release  or  to switch between **am** and **pm**.

Press and release  to set **am** or **pm**.

6

The bell symbol, **OFF**, and the flashing **A-2** appear on the display.

Either set the **A-2** test reminder or press and release  to set up more options.

7

To exit, press and **hold**  until the flashing test strip symbol appears on the display.

Setting the Hypoglycemic (Hypo) Alert

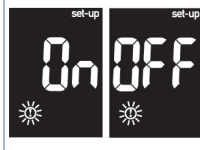
WARNING

- This function is no substitute for hypoglycemia training by your healthcare professional.
- Before you set the hypo alert, talk to your healthcare professional to help you decide what blood glucose level is your hypoglycemic level.

1

Turn the meter on.
The flashing test strip symbol appears on the display.
Press and **hold**  until **set-up** appears on the display.

2

Press and release  repeatedly until **set-up**, **OFF**, and the flashing  appear.

3

Press and release  or  to switch between **On** and **OFF**.
Press and release  to set your choice.
If you select **On**, **set-up** appears and  flashes.

4

Press and release  or  to adjust the level.
Press and release  to set the level.
To exit, press and **hold**  until the flashing test strip symbol appears on the display.
The set-up mode is complete.

5 Meter Memory, Setup, and Data Transfer

Transferring Results to a Computer

We offer a variety of software to help you transfer your results. For information on Accu-Chek software, contact the Accu-Chek Customer Care Service Center at 1-800-858-8072.

You can transfer your stored results to a computer to track, identify patterns, and print.

Transferring Data Directly to a Computer Using Specialized Software and an Infrared Cable

1. Install the software according to the instructions.
2. To transfer the results to a computer, connect the infrared cable according to the instructions.
3. Run the software program and follow the instructions for data transfer. Make sure the software is ready to accept data from the meter.
4. With the meter off, press and **hold** both  and  until 2 arrows on the display alternately flash.
5. Locate the infrared (IR) window on the top of the meter.
6. Locate the IR window on the infrared cable.
7. Place the meter on a flat surface. Point the 2 IR windows toward each other. They should be 1–4 inches apart.
8. Do not move the infrared cable or meter during the data transfer.
9. Follow the prompts on the software.
10. The software program may shut off the meter automatically when the data transfer is complete. Should this occur, follow the prompts on the display.

NOTE

- If the data did not transfer successfully, try again. Contact the Accu-Chek Customer Care Service Center at 1-800-858-8072 if you still have problems.
- To make the most of the transfer feature, make sure the meter is set to the correct time and date.

What is the difference between cleaning and disinfecting?

Cleaning is the removal of dirt from the meter or lancing device.³

Disinfecting is the removal of most, but not all, disease-causing and other types of microorganisms (bloodborne pathogens) from the meter or lancing device.³

Approved Cleaning and Disinfecting Product

The following product has been approved for cleaning and disinfecting the meter and lancing device:

Super Sani-Cloth (EPA* reg. no. 9480-4)

Super Sani-Cloth can be purchased from Amazon.com, Officedepot.com and Walmart.com.

- Do not use any other cleaning or disinfecting solutions. Using solutions other than the Super Sani-Cloth could result in damage to the meter and lancing device.
- The effect of using more than one product interchangeably to clean and disinfect the meter and lancing device has not been tested. Always use Super Sani-Cloth to clean and disinfect the meter and lancing device.
- Roche has tested the approved product for a total of 520 cycles, which is equal to cleaning and disinfecting once per week for a 5-year period.

*Environmental Protection Agency

NOTE

For technical assistance or questions on cleaning and disinfecting, contact the Accu-Chek Customer Care Service Center at 1-800-858-8072.

Cleaning and Disinfecting the Meter

To clean and disinfect without damaging the meter, follow these procedures carefully.

When to Clean and Disinfect the Meter

- Clean the meter to remove visible dirt or other material prior to disinfecting.
- Clean and disinfect the meter at least once per week and when blood is present on the surface of the meter.
- Clean and disinfect the meter before allowing anyone else to handle the meter. Do not allow anyone else to use the meter on themselves for testing purposes.

NOTE

If you notice any of the following signs of deterioration after cleaning and disinfecting the meter, contact the Accu-Chek Customer Care Service Center at 1-800-858-8072: Residue around buttons, clouding of display, or button malfunction.

What to Clean and Disinfect

The following parts of the meter should be cleaned and disinfected:

- The area around slots and openings (do not get any moisture in slots or openings)
- The meter display
- The entire meter surface

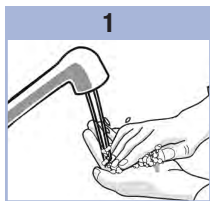
How to Clean and Disinfect the Meter


WARNING

Failure to follow these instructions will damage the meter and stop it from working properly.


- DO NOT clean or disinfect the meter while performing a blood glucose or control test.
- DO NOT get any moisture in slots or openings.
- DO NOT spray anything onto the meter.
- DO NOT immerse the meter in liquid.
- Always use the same product for both cleaning and disinfecting.

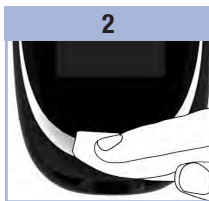
1
Wash hands thoroughly with soap and water. Turn the meter off and wipe the entire meter surface with an approved cleaning and disinfecting product.

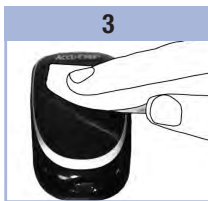2
Carefully wipe around the test strip slot and other openings. Make sure that no liquid enters any slot or opening.

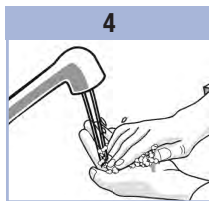3
Dry the meter with a soft cloth or gauze. Make sure that no solution is seen in any slot or opening.

4
For disinfecting the meter, get a new wipe or cloth. Repeat steps 2 and 3 following the manufacturer's instructions for disinfecting. Wash hands thoroughly with soap and water.

Cleaning and Disinfecting the Lancing Device

To clean and disinfect without damaging the lancing device, follow these procedures carefully.

When to Clean and Disinfect the Lancing Device

- Clean the lancing device to remove visible dirt or other material prior to disinfecting.
- Clean and disinfect the lancing device at least once per week to remove visible dirt or other material for safe handling.
- Clean and disinfect the lancing device before allowing anyone else to handle the lancing device, for instance, if you have someone assisting you. Do not allow anyone else to use the lancing device.

NOTE

- Do not throw away the cap after each use. Use the approved cleaning and disinfecting product on the cap.
- Remove the lancet drum before cleaning and disinfecting the lancing device.
- If you notice any residue around the buttons after cleaning and disinfecting the lancing device, contact the Accu-Chek Customer Care Service Center at 1-800-858-8072.
- You might observe a slight discoloration of the lancing device after multiple cleaning and disinfecting cycles. This does not affect the functionality of the lancing device.

What to Clean and Disinfect

The following parts of the lancing device should be cleaned and disinfectd:

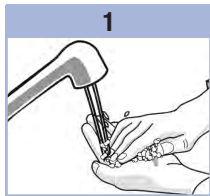
- The entire lancing device surface
- The cap

How to Clean and Disinfect the Lancing Device


WARNING

Failure to follow these instructions may damage the lancing device and stop it from working properly.

- DO NOT get any moisture into any openings.
- Always use the same product for both cleaning and disinfecting.

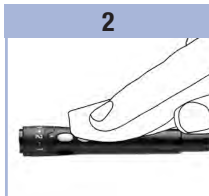
Wash hands thoroughly with soap and water.

Wipe the entire surface of the lancing device and the inside of the cap with an approved cleaning and disinfecting product.

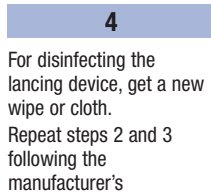Dry the lancing device and cap with a soft cloth or gauze. Make sure no solution is seen in any opening.

For disinfecting the lancing device, get a new wipe or cloth. Repeat steps 2 and 3 following the manufacturer's instructions for disinfecting. Wash hands thoroughly with soap and water.

Meter Maintenance

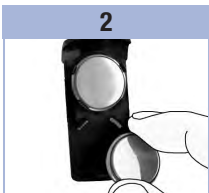
The meter automatically tests its own systems every time you turn it on and lets you know if something is wrong. See Chapter 7, Display and Error Messages.

If you have problems with the meter or think the results are not accurate, perform a control test with an unexpired test strip and control solution. If the control result is not within the acceptable range, contact the Accu-Chek Customer Care Service Center at 1-800-858-8072.

Changing the Batteries

Use your thumb to slide the battery drawer out of the meter.

Remove the old batteries and place the new ones in the battery drawer with the (+) side facing down.

Slide the battery drawer back into position until it locks into place.

NOTE

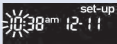
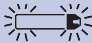
- The meter uses two 3-volt batteries, coin cell type CR2032. This type of battery can be found in many stores. It is a good idea to have spare batteries available.
- Always replace both batteries at the same time and with the same brand.

Display and Error Messages

WARNING

Never make therapy decisions based on an error message.

If you have any concerns or see any other error display, contact the Accu-Chek Customer Care Service Center at 1-800-858-8072.

Display	Action
The meter will not turn on or the display is blank.	<ul style="list-style-type: none"> Batteries are dead. Insert new batteries. Display is damaged. Contact the Accu-Chek Customer Care Service Center. Meter is defective. Contact the Accu-Chek Customer Care Service Center. Extreme temperatures. Move the meter to a more temperate area.
	Battery power is low. Change the batteries soon.
	The meter is in set-up mode, waiting for you to change or confirm settings.
	The meter is ready for you to insert a test strip.
	The meter is ready for a drop of blood or control solution.
HI	Blood glucose may be higher than the measurement range of the system. See Chapter 4, Unusual Blood Glucose Results.
LO	Blood glucose may be lower than the measurement range of the system. See Chapter 4, Unusual Blood Glucose Results.
*	A general marker was assigned to this test result.
	Blood glucose is below the defined hypoglycemic (low blood glucose) level. See Chapter 4, Unusual Blood Glucose Results.
	A pre-meal marker was assigned to this test result.

7 Maintenance and Troubleshooting

Display	Action
	A post-meal marker was assigned to this test result.
	A pre-meal marker was assigned to this test result and the post-meal test reminder has been activated.
E-1	The test strip may be damaged or not properly inserted. Remove and reinsert the test strip or replace it if damaged.
E-3	Your blood glucose may be extremely high or a meter or a test strip error has occurred. <ul style="list-style-type: none">• If your test result matches how you feel, contact your healthcare professional immediately.• If your test result does not match how you feel, repeat the blood glucose test. See Chapter 4, Unusual Blood Glucose Results.<ul style="list-style-type: none">• If the E-3 code still appears for your blood glucose test, your blood glucose result may be extremely high and above the system's reading range. Contact your healthcare professional immediately.• If the second test result does not match how you feel, perform a control test with the control solution and a new test strip.<ul style="list-style-type: none">• If the control result is within the acceptable range, review the proper testing procedure and repeat the blood glucose test with a new test strip.• If the control result is not within the acceptable range, see Chapter 2, Understanding Out-of-Range Control Results.
E-4	Not enough blood or control solution was drawn into the test strip for measurement or was applied after the test had started. Discard the test strip and repeat the test.
E-6	Blood or control solution was applied to the test strip before the flashing drop symbol appeared on the display. Discard the test strip and repeat the test.
E-7	An electronic error occurred, or in rare cases, a used test strip was removed and reinserted. Turn the meter off and on, or take the batteries out for 20 seconds and reinsert them. Perform a blood glucose or control test. If the problem persists, contact the Accu-Chek Customer Care Service Center.

Display	Action
E-8	The temperature is above or below the proper range for the system. Refer to the test strip package insert for system operating conditions. Move to an area with the appropriate conditions, wait 5 minutes, and repeat the blood glucose or control test. Do not artificially heat or cool the meter.
E-9	The batteries are almost out of power. Change the batteries now. If the message reappears after the batteries have been replaced, reset the meter. To reset the meter, slide the battery drawer out of the meter, press any meter button, then reinsert the battery drawer back into position.
E-10	The time and date settings may be incorrect. Make sure the time and date are correct and adjust, if necessary.

Product Limitations

See the literature packaged with the test strips and control solution for the latest information on product specifications and limitations.

Specifications	
Blood volume Sample type Measuring time Measurement range Test strip storage conditions System operating conditions Relative humidity operating range	Refer to the test strip package insert.
Meter storage conditions	Temperature: -13–158 °F
Memory capacity	500 blood glucose results and 20 control results with time and date
Automatic power off	2 minutes
Power supply	Two 3-volt lithium batteries (coin cell type CR2032)
Display	LCD
Dimensions	69 × 43 × 20 mm (LWH)
Weight	Approx. 40 g (with batteries)
Construction	Hand-held
Protection class	III
Meter type	The Accu-Chek Nano meter is suitable for continuous operation.
Control solution storage conditions	Refer to the control solution package insert.

Electromagnetic Compatibility – This meter meets the electromagnetic immunity requirements as per EN ISO 15197 Annex A. The chosen basis for electrostatic discharge immunity testing was basic standard IEC 61000-4-2. In addition, the meter meets the electromagnetic emissions requirements as per EN 61326. The meter's electromagnetic emission is thus low. Interference from the meter to other electrically-driven equipment is not anticipated.

Performance Analysis – Refer to the test strip package insert.

Test Principle – Refer to the test strip package insert.

Product Safety Information

WARNING

- Strong electromagnetic fields may interfere with the proper operation of the meter. Do not use the meter close to sources of strong electromagnetic radiation.
- To avoid electrostatic discharge, do not use the meter in a very dry environment, especially one in which synthetic materials are present.

Travel Documentation

If you are traveling on a commercial airline, you may be required to provide documentation certifying that this meter meets environmental conditions and test procedures for Airborne Equipment (RTCA/DO-160F) section 21 Emission of Radio Frequency Energy. Visit accu-chek.com or contact the Accu-Chek Customer Care Service Center at 1-800-858-8072 to obtain a copy of the document.

Discarding the Meter, Test Strips, Lancing Devices, Lancets, and Batteries

WARNING

- Any product coming in contact with blood is considered contaminated (potentially infectious).*
- During normal testing, any blood glucose meter may come in contact with blood.
- Lancing devices may also be considered sharps. Disposal of sharps is regulated by law in many jurisdictions.

The European Union has a requirement for improving waste management practices for certain electronic equipment, but meters fall outside the scope of the European Directive 2002/96/EC.** This is not a requirement for the United States; however, Roche is committed to recycling and sustainability. Comply with any laws or ordinances relating to the disposal of sharps and/or contaminated products. Contact your local health department or other appropriate authorities for proper handling and disposal of used meters, used test strips, used lancets, and used batteries. Consider the following points when discarding used testing materials:

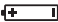
Consider recycling the meters and batteries at an appropriate facility. Be aware the meter is potentially hazardous electronics scrap (e-scrap) and should be disposed of accordingly. The batteries are potentially hazardous also and should be discarded accordingly.

Disinfect the meter before recycling or discarding.

*29 CFR 1910.1030 – Bloodborne pathogens

**Directive 2002/96/EC – Directive on waste electrical and electronic equipment (WEEE)

Explanation of Symbols

	Caution, refer to safety-related notes in the instructions for use accompanying this product.
	3-volt coin cell type CR2032
	Global Trade Item Number
	Serial Number

Warranty

Accu-Chek Nano Meter 30-day Money-back Guarantee for Qualifying Consumers

Roche offers qualifying consumers that purchase an Accu-Chek Nano meter, a 30-day money back guarantee. If you are not fully satisfied with your Accu-Chek Nano meter, contact the Accu-Chek Customer Care Service Center toll-free at 1-800-858-8072 to determine whether you qualify to receive a full refund within 30 days of purchase. If you are covered under Medicare, Medicaid, other federal/state programs, or private insurance you are NOT eligible for this money-back offer. Consumers affected by this exclusion may instead request a different Accu-Chek meter/system. The refund will be limited to the amount paid by you net of any rebates. You must have a copy of the dated itemized purchase receipt and the original packaging to obtain this refund.

Accu-Chek Nano Meter Limited 3-Year Warranty

Roche warrants to the original purchaser of the meter that your Accu-Chek Nano meter will be free from defects in materials and workmanship for three years from the date of purchase. If, during this 3-year period, the meter does not work properly because of a defect in materials or workmanship, Roche will replace it with a new Accu-Chek Nano meter or equivalent product free of charge. The warranty on the replacement meter will expire on the date of the original warranty expiration or 90 days after the shipment of a replacement system, whichever period is longer. The purchaser's exclusive remedy with respect to the Accu-Chek Nano meter shall be replacement.

This warranty does not apply to the performance of an Accu-Chek Nano meter that has been damaged by accident or has been altered, misused, tampered with, or abused in any way. Roche will handle meters that show damage or abuse according to its Non-Warranty Service Policy described on the following page.

THE ABOVE WARRANTY IS EXCLUSIVE OF ALL OTHER WARRANTIES, AND ROCHE MAKES NO OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING WITHOUT LIMITATION, THE IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IN NO EVENT SHALL ROCHE BE LIABLE TO THE PURCHASER OR ANY OTHER PERSON FOR ANY INCIDENTAL, CONSEQUENTIAL, INDIRECT, SPECIAL, OR PUNITIVE DAMAGES ARISING FROM OR IN ANY WAY CONNECTED WITH THE PURCHASE OR OPERATION OF THE METER OR ITS PARTS. NO WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, IF ANY IS IMPLIED FROM THE SALE OF THE METER, SHALL EXTEND FOR A LONGER DURATION THAN THREE YEARS FROM THE DATE OF PURCHASE.

Some states do not allow limitations on how long an implied warranty will last or the exclusion of incidental or consequential damages, so the above limitation and exclusion may not apply to you. This warranty gives you specific legal rights, which vary from state to state.

Non-Warranty Service Policy

Roche Non-Warranty Service Policy applies to meters where the above warranty has not become effective, has become inapplicable, or has expired. Roche will replace, at its option, meters returned to it for a service charge (not to exceed \$35). Replacement will be with the same or similar product. Replacement meters will be warranted for a period of 90 days from shipment under a limited warranty providing for replacement of parts and labor at no charge.

Warranty and Service Instructions

All requests for return of Accu-Chek Nano meters under the above warranty or service policy must be made to the Accu-Chek Customer Care Service Center. You will be mailed a return authorization label, which must be affixed to your carton for shipping the system to Roche. Cartons received without this label will be returned to you at your expense.

Customers experiencing difficulties should review the troubleshooting information in Chapter 7 of this booklet. Further inquiries should be directed to the Accu-Chek Customer Care Service Center.

Be sure to fill out and mail the Warranty Card that comes with the Accu-Chek Nano SmartView system.

Limited License

WARNING

CAUTION – A RESTRICTED LICENSE LIMITS USE OF THE ACCU-CHEK NANO SMARTVIEW SYSTEM – READ CAREFULLY THE LIMITATIONS RECITED BELOW.

The Accu-Chek Nano SmartView system (meter and test strips) and its use are protected by U.S. Patent Nos. 7,276,146 (expires 4-October-2022); 7,276,147 (expires 4-October-2022); 7,452,457 (expires 2-May-2026); 7,488,601 (expires 1-February-2026); 7,569,126 (expires 28-December-2026); and 7,604,721 (expires 12-August-2026). A license to use the Accu-Chek Nano SmartView system is required until the last-to-expire patent listed above and is only granted when the Accu-Chek Nano meter is used with the Accu-Chek SmartView test strips.

Accu-Chek SmartView test strips are specifically manufactured for operation with the Accu-Chek Nano meter. Use of other test strips supplied by another manufacturer may prevent or impair the proper function of the Accu-Chek Nano SmartView system. Using the Accu-Chek Nano SmartView system indicates your acceptance of the restricted license to use the Accu-Chek Nano meter only with Accu-Chek SmartView test strips. Further, if you have purchased an Accu-Chek Nano SmartView blood glucose monitoring kit or an Accu-Chek Nano meter that includes this restricted license, then this restricted license applies regardless of any additional offers found in Accu-Chek SmartView test strip packages. If you do not agree to the terms and conditions of the restricted license, you may return, at the place of purchase, the unused Accu-Chek Nano SmartView system for a full refund. If you have any questions, contact the Accu-Chek Customer Care Services Center at 1-800-858-8072.

Except where prohibited by statute, all warranties covering the Accu-Chek Nano SmartView system are voided by use of the Accu-Chek Nano meter with any test strips other than Accu-Chek SmartView test strips.

WARNING

A RESTRICTED LICENSE LIMITS USE OF THE ACCU-CHEK FASTCLIX SYSTEM (lancing device and lancet drums). READ CAREFULLY THE LIMITATIONS RECITED BELOW.

The Accu-Chek FastClix system (device and lancet drums) and its use are protected by U.S. Patent Nos. 7,785,338 (expires 5-January-2026). A license to use the Accu-Chek FastClix system is required until the expiration of the last-to-expire patent listed above and is only granted when Accu-Chek FastClix lancet drums are used with the Accu-Chek FastClix device.

Accu-Chek FastClix lancet drums are high precision components that are produced to the close tolerances required for successful operation with the Accu-Chek FastClix device. Use of other lancet drums with the Accu-Chek FastClix device may prevent or impair proper function of the Accu-Chek FastClix device.

Using the Accu-Chek FastClix device indicates your acceptance of the restricted license to use the Accu-Chek FastClix device only with Accu-Chek FastClix lancet drums. Further, if you have purchased an Accu-Chek FastClix device that includes this restricted license, then this restricted license applies regardless of any additional offers found in Accu-Chek FastClix device packages. If you do not agree to the terms and conditions of the restricted license, you may return, at the place of purchase, the unused Accu-Chek FastClix device for a full refund. If you have any questions, contact the Accu-Chek Customer Care Services Center at 1-800-858-8072.

Except where prohibited by statute, all warranties covering the Accu-Chek FastClix device are voided by use of the Accu-Chek FastClix device with any lancet drums other than Accu-Chek FastClix lancet drums.

Patent Information

U.S. Pat.: <http://www.roche-diagnostics.us/patents>

Additional Supplies

Test Strips: Accu-Chek SmartView test strips

Control Solution: Accu-Chek SmartView control solution

Lancets: Accu-Chek FastClix 102-ct lancet drums (17-6 ct. drums)

AST Cap: Available upon request from Accu-Chek Customer Care Service Center at 1-800-858-8072.

References

- ¹ FDA Public Health Notification: "Use of Fingerstick Devices on More than One Person Poses Risk for Transmitting Bloodborne Pathogens: Initial Communication, (2010). Update 11/29/2010." <http://www.fda.gov/MedicalDevices/Safety/AlertsandNotices/ucm224025.htm>. Accessed February 2, 2012.
- ² CDC Clinical Reminder: "Use of Fingerstick Devices on More than One Person Poses Risk for Transmitting Bloodborne Pathogens, (2010)." <http://www.cdc.gov/injectionsafety/Fingerstick-DevicesBGM.html>. Accessed February 2, 2012.
- ³ Healthcare Infection Control Practices Advisory Committee (HICPAC), William A. Rutala, Ph.D., M.P.H., and David J. Weber, M.D., M.P.H. Centers for Disease Control and Prevention, 2008. "Guideline for Disinfection and Sterilization in Healthcare Facilities." Atlanta.

A

alternate site testing 20

B

batteries, changing 38

battery type 42

blood glucose results, unusual 22

blood glucose, test 18

button, power/set 9

C

comfort dial 13

computer, transferring results to 32

control results, understanding out-of-range 12

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